

*MR Dr. med. Klaus Graubner*  
*Facharzt für Gynäkologie und Geburtshilfe*

Breitscheidstraße 1a  
39517 Tangerhütte

☎ (0 39 35) 2 83 97  
eMail: klaus.graubner@t-online.de

Fax: (0 39 35) 21 30 06  
Internet: www.klaus-graubner.de



---

MR Dr. med. Klaus Graubner • Breitscheidstraße 1a • D-39517 Tangerhütte

Meyer-Haake  
Medical Innovations  
Am Joseph 9

61273 Wehrheim

---

Ihr Zeichen, Ihre Nachricht vom

Unser Zeichen, unsere Nachricht vom

Datum

Tangerhütte, 20.07.2005

## **BIO-CONE – Electrode for Conisation**

Dear Mrs. Meyer-Haake,

today I would like to answer your friendly mails.

I'm using an ERBE ICC 200 (120 watts) for cutting level 1 and 60 watts for coagulating.

The advantage of the **BIO-CONE-Electrode for Conisation** compared to the loop-electrodes is to find in the fact that an exact rotation of the electrode is possible due to the fixation by means of the thorns. Furthermore is to mention that the very fine and relatively short wire allows a very fast and precise cut. The result is an exact conus, because one doesn't "get stuck" as you do when using the loop electrodes. Due to the exact conus-shape less cervical tissue is removed, as it is the case when using the loop electrodes. The cosmetic result of conisation is extremely well. Most of the time you won't even see that a conisation has been performed.

The extracted conus can be histopathologically examined up to the wound edges without any problems.

Kind regards



Dr. Klaus Graubner